CLASSIC SR

ACRILATO FD

- a. Instrucciones de uso en Español (ES)
- b. Instructions for use in English (EN)
- c. Instruções de uso em Português (PT)
- d. Mode d'emploi en français (FR)
- e. Istruzioni per l'uso in Italiano (IT)
- f. Инструкция по применению на английском языке (RU)
- g. Türkçe kullanım talimatları (TR)
- h. Uputstvo za upotrebu na srpskom (SR)
- i. Instrukcja obsługi w języku polskim (PL)
- j. Návod k použití v češtině (CS)
- k. Gebrauchsanweisung in Deutsch (DE)

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1-PRODUCT DESCRIPTION

CLASSIC SR

Polyamide for the manufacture of denture bases

ACRILATO FD

PMMA for the manufacture of denture bases

2-INDICATIONS / COUNTER-INDICATIONS / LIMITATIONS

CLASSIC SR

Indicated for partial, full and combined prostheses with chrome-cobalt.

Recommended indication: partial dentures.

ACRILATO FD

Indicated for partial, full and combined prostheses with chrome-cobalt.

Recommended indication: full dentures.

3-SYMBOLS USED

	S = Small M = Medium L = Large		
	Net weight	Diameter	
	Injection temperature		
	T1= Injection or plasticized time T2=Injection holding time		
	Air pressure		
	Unique Device Identifier		

	Lot number Expiration/expiry date Manufacturing date
	Manufacturer
	Prohibited reuse Protect from the sun's rays Keep in a cool and dry place
	Do not use if packaging is damaged or defective Follow the instructions
	Authorised representative
	Medical Device

4-COMMERCIAL PRESENTATION

CLASSIC SR

MEASURES AND WEIGHTS

Small ø 22x55mm / 13,7g
 Medium ø 22x85mm / 19,9g
 Large ø 22x120mm / 27,4g
 Small ø 25x50mm / 14,2g
 Medium ø 25x70mm / 19,6g
 Large ø 25x90mm / 25,8g

AVAILABLE COLORS

Pale Pink
 Medium pink
 Dark pink
 Transparent

ACRILATO FD

MEASURES AND WEIGHTS

Medium ø 22x85mm / 24g
 Large ø 22x120mm / 32,8g
 Medium ø 25x70mm / 24,5g
 Large ø 25x90mm / 31,3g

AVAILABLE COLORS

Medium pink
 Transparent
 Soft pink
 Translucent soft pink

Primary packaging: Aluminium tubes 22 mm and 25 mm in diameter

Secondary packaging: Plastic tubes or carton boxes, with moisture-absorbent material

Pre-dried

Note: Measurement and weight values are nominal and may differ slightly from cartridge to cartridge.

5-CARTRIDGE HANDLING

Open the plastic tube or the carton box, then remove the plastic cap from the aluminum tube.

Inject the cartridge with the lot number facing downward, not facing upward toward the muffle.

6-SAFETY INFORMATION

ALLERGENIC INFORMATION

Potential Known Allergenic Substances present in the product: None

This product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product (product sample supplied upon request for allergy test of allergy sufferers). **(S1)**

PEDIATRIC USE

The use of a denture base in children should be prescribed by the Pediatric Dentist. **(S2)**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

7-EXPIRY / STORAGE

Store in cool dry place. **(S3)**

Do not expose to the rays of the sun.

Do not use if the container is damaged or defective. **(S4)**

Use immediately after opening. **(S5)**

Expiration / expiration date / Shelf life: 24 months

8-MAKING OF THE DENTURE

8.1 MODEL CASTING

It is recommended to make the models in special plaster type IV (Densita).

8.2 PARALLELIZATION OF THE MODEL

In the same way as with chrome, it is recommended to use a pendular parallelometer to look for the most favorable insertion axle, determining the dental equator, according to which the retainers we will be designed.

Then, proceed to eliminate all the inconvenient or blind spots, applying wax and passing the rod. **(S6)**

8.3 RETENTIVITY

Once the parallelization has been completed, tidy up any wax excess that may have been left over the dental equator and the gingival pad, from the medial area until the retainer's end. This will give the dentures the necessary retentivity. (S7)

8.4 RELIEFS

Perform reliefs in wax covering the cervical areas.

8.5 DUPLICATE

Make a duplicate of the model to work on hereafter.

IMPORTANT!

Perform the casting with plaster type IV (Densita), which is able to withstand the pressure to be used in the injection. (S8)

8.6 OCCLUSION PAD

Once the working model has been completed, we will make the occlusion pad.

8.7 SPECIAL INSTRUCTIONS FOR BONDING TO SYNTHETIC TEETH

The use of mechanical retention as described below is required to obtain proper bonding of synthetic teeth to the denture base material. (S9)

8.8 DEFINITIVE WAXING

Minimum recommended thickness for the base plaque: (S10)

- ✓ Upper dentures = 1.5 mm
- ✓ Lower dentures = 3 mm

For the retainers to have resistance, good memory and aesthetics, it is suggested that they are applied on the third part of the gingival area (tooth's retentive area), covering part of the gums (applies to CLASSIC SR).

In order to be injected, the retainers require a minimum of 1.5 mm.

Occlusal supports

For the occlusal supports to be resistant, they must be wider and deeper than the ones performed with metal. Recommended depth = 1.3 mm.

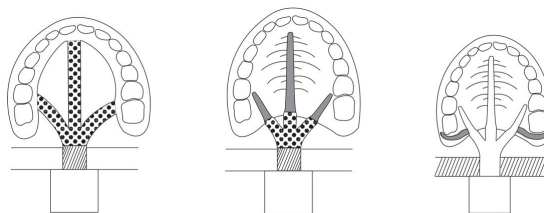
8.9 PLACING IN THE MUFFLE

Scrub the retentive areas of the plaster teeth.

Use special plaster type IV

8.10 INJECTION CONDUITS (S11)

Perform tubes in wax. The tubes must follow a straight path on the material intake plane during the injection. For this reason, the model must be placed on that same plane; it must neither be buried nor be much higher than the muffle plane.

Main sprues: 6 mm of diameter.

They start at the very entrance of the muffle in such a way that the material is distributed among the main sprues as soon as it gets into the muffle. These sprues must rest on the muffle plaster, they are not aerial.

Secondary or auxiliary sprues: 3 mm of diameter.

Optionals. For upper Dentures and areas which are difficult to reach for the material. Secondary sprues are placed on the waxing. They must never be aerial.

Direct the lateral main sprues towards molar / premolar, in a "V" shape, and not in a "T" shape.

8.11 COUNTER MUFFLE

Put plaster separator. Present counter muffle and screw. Fill with 255 gr of special plaster type IV

8.12 MUFFLE CLEANSING

Loosen the screws two turns before cleansing. Immersion time: from 6 to 8 minutes.

8.13 TUBES CORRECTION – TEETH PERFORATIONS’ CLEANSING

Once the muffle has been cleansed, we must rectify and correct all the tubes, eliminating any thin layer or excess that may have remained.

8.14 SEPARATOR FOR ACRYLIC

Let the muffle cool down before applying the separator.

2 or 3 layers must be applied, according to the kind of separator used. It's important to let it dry out completely between one layer and the next one.

8.15 PREPARATION OF THE MUFFLE FOR THE INJECTION

Put the screws on the counter muffle side, and tighten them together with the one diagonally opposed.

8.16 INJECTION PARAMETERS

Apply Deflex silicone release agent or similar. Inject the cartridge with the lot number facing down.

CLASSIC	MD 22 TOUCH	MD 22	MD 25 TOUCH	MD 25	MAD1300 TOUCH
	280°C +/- 5°C	280°C +/- 5°C	280°C +/- 5°C	280°C +/- 5°C	280°C +/- 5°C
 T1	15' min	15' min	20' min	20' min	15' min
 T2	1' min	1' min	1' min	1' min	1' min
	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	6 / 7 kg/cm2 = 86 / 100 PSI

ACRILATO	MD 22 TOUCH	MD 22	MD 25 TOUCH	MD 25	MAD1300 TOUCH
	265°C +/- 5°C	265°C +/- 5°C	265°C +/- 5°C	265°C +/- 5°C	265°C +/- 5°C
 T1	15' min	15' min	20' min	20' min	15' min
 T2	1' min	1' min	1' min	1' min	1' min
	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	7 / 8 kg/cm2 = 100 / 114 PSI	6 / 7 kg/cm2 = 86 / 100 PSI

NOTE: Injection parameters indicated for Deflex injection machine. For other injectors, consult the manufacturer or supplier of the equipment.

8.17 RETOUCHING AND POLISHING

Tubes cutting: double-sided diamond steel disc.

Thick retouching: Tungsten Carbide bur (red ring cross multifil).

Remove excesses: steel brush (fine and soft threads), or abrasive cloth.

Fine retouching: abrasive rubbers and / or water sandpaper N° 240 and N° 600.

Polishing: water sandpaper N° 1000 and N° 1500 and / or, pumice stone.

Brightness: Deflex high brightness paste and fabric wheel.

9-OBSERVATIONS FOR THE USER

9.1 DENTURE HYDRATION

Submerge the denture without cutting the injection tubes and without the model for 15 minutes in boiling water, to release the tensions of the thermo-injection. **(S12)**

9.2 REPAIRS

Repairs to prostheses made of Deflex CLASSIC SR material must always have mechanical retention by means of perforations **(S13)**, unless some type of adhesive is used that makes mechanical retention unnecessary.

Before using an adhesive to repair a dental prosthesis, the use information provided by the adhesive manufacturer should be read to verify that the adhesive is suitable for permanent contact with the oral mucosa. **(S14)**

9.3 FINAL DISPOSAL OF WASTE

Contact your distributor or waste management local authorities to ensure proper disposal of the waste originated during the injection process. **(S35)**

9.4 OTHER OBSERVATIONS

For professional use only.

The technical steps outlined for the processing of this product are those indicated and / or suggested by the manufacturer, and correspond to the current state of the art developed by the manufacturer.

The indications related to the injection flask refer to the Deflex brand flask.

The use of complementary techniques, materials, and equipment for processing the product is the responsibility of the user. In cases of unsuccessful results, the manufacturer limits the rights to compensation for damages exclusively to the commercial value of its products.

Do not use toothpaste or any other abrasive product since they may scratch the surface of your denture. Do not use alcohol for cleaning.

10- CONTACT DETAILS

Technical department: info@deflex.com.ar

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